



Elipsa provides a soothing, relaxing, non-invasive, and therapeutic treatment that can be administered on its own or combined with many other treatments and services to elevate results. All LightStim devices have been shown to provide clinically proven results. Most of the benefits below may be seen after just one treatment.

Benefits Associated with LightStim Elipsa


- + Minimizes the appearance of pores, fine lines, and wrinkles
- + Helps energize and restore balance for optimal skin function
- + Promotes a more even skin tone
- + Calms and reduces redness
- + Produces a radiant glow
- + Works synergistically with other treatments and ingredients for better results
- + Increases blood circulation and oxygenation
- + Helps to accelerate recovery
- + Destroys acne causing bacteria

Elipsa Wrinkle Mode

FDA Cleared to treat wrinkles on the entire face

Elipsa Acne Mode

FDA Cleared to treat mild to moderate and inflammatory acne

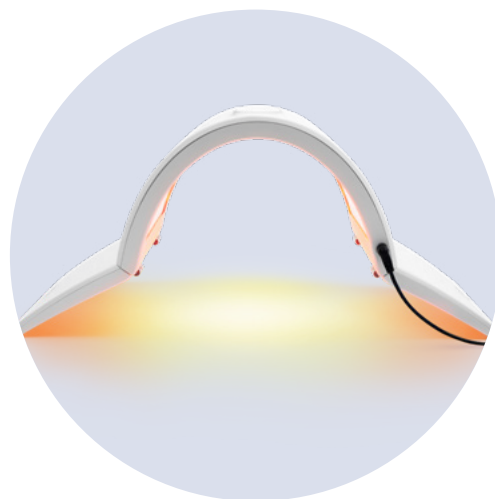

Combination Treatments

LightStim Elipsa may be used in conjunction with many other treatments and services to promote benefits and/or calm the skin post-procedure:

- | | | |
|---|---------------------|---|
| + Facials | + Microdermabrasion | + Laser and IPL |
| + HydraFacial® | + Dermaplaning | • When to introduce the gentle warmth of LightStim after treatment is strictly at the discretion of the laser tech or medical professional. |
| + Enzyme Treatments / Peels | + Microcurrent | + Radiofrequency (RF) |
| • Because formulations vary, consult with peel manufacturer to determine if a waiting period is necessary after peel and before using a mildly warm LED device on the skin. When in doubt LightStim can be used prior to the treatment. | + Waxing & Sugaring | + Surgery |
| | + Oxygen Treatment | + Post-Procedure Recovery |
| | + Microblading | + Acupuncture |
| | + Microneedling | + IV Supplementation |
| | + Dermal Injections | |



Ingredient Guidelines

LightStim devices may be used on clean, dry skin or in conjunction with topical products. LightStim's unique technology works in harmony with high performance ingredients to enhance results.

- + Use LightStim PhotoSerum, LightStim SheetMasque or the products of your choice. When choosing products always consider skin type and condition.
- + When in doubt as to whether a topical product should be applied prior to or after LightStim treatment, ask the product manufacturer if it's safe to use a warm LED device on the skin after application of their product and/or simply apply product after LightStim treatment.
- + Ingredient categories that may be used prior to LightStim treatment (if non-irritating and safe to use with warmth) include collagen building peptides, cell stimulators, anti-inflammatories, antioxidants, and hydrators.
- + Ingredient categories to avoid prior to LightStim but that may be used after treatment include anything that may pose a risk of irritation or dryness, products that should not be exposed to warmth, Retin-A, benzoyl peroxide, harsh acids, potentially reactive forms of vitamin C and A, products containing metals (like colloidal silver and copper peptides), opaque masks and/or highly occlusive ingredients.

LightStim is pleased to partner with some of the industry's most reputable product lines to create step-by-step protocols designed to boost results. Please contact LightStim for more information.

LightStim Precautions

- + Extreme sensitivity to light – migraines brought on by bright light
- + Use of photosensitizing drugs
- + Melasma or hyperpigmentation exacerbated by mild warmth
- + Do not apply products that should not be exposed to mild warmth prior to LightStim treatment
- + Accutane should be discontinued for 6 months in advance of treatment
- + Clients who are pregnant must seek approval from a medical professional
- + Clients with suspicious lesions or skin cancers should consult a medical professional
- + Clients who are being treated for any type of serious health condition should seek approval from a medical professional before undergoing LED therapy





Instructions for Use

1. Plug into a wall outlet
2. Press any of the buttons to wake up the device if the time display is off
3. Select the treatment mode indicated by the color on the time display
 - Red – Wrinkles
 - Blue – Acne
4. Select treatment time by using the 1 min and 5 min buttons
 - Recommended treatment time is 16 minutes in wrinkle mode and 12 minutes in acne mode
5. Select the power button to start treatment
6. Light will turn off once treatment is complete

Elipsa Treatment Room Protocol

1. Cleanse
2. Exfoliate (as indicated)
3. Tone (if desired)
4. Perform extractions (as necessary)
5. Apply treatment serums or transparent/near transparent mask, if desired (refer to Ingredient Guidelines in this document)
6. Position LightStim provided goggles over the client's eyes and ensure the elastic band is in place snugly but comfortably around the head – if necessary moistened cotton rounds may be used under the goggles
7. Before moving Elipsa into position over the face, turn the device on – this will provide an opportunity for the eyes to adjust to the brightness (see Instructions for Use above)
8. Position Elipsa approximately 1" from the treatment area by using the adjustable legs to increase or decrease the height
9. When the treatment is complete, remove the device
10. Remove mask (if applied)
11. Apply moisturizer and sunscreen

LightStim & Oncology

LightStim is proud to be a Mission Partner with Oncology Spa Solutions. We strongly recommend skin care professionals are thoroughly trained before treating any client affected by cancer. LED can be a valuable tool under these circumstances, however, it may be advisable to modify protocols as outlined above.

For more information, please contact Becky Kuehn at becky@oncologyspasolutions.com.

Care & Cleaning

Acrylic Covering LEDs

You may disinfect the acrylic after each use. Wipe the acrylic surface of the device with the approved disinfectant, CaviWipes Towelettes. Do not use alcohol or other non-approved cleaning products on the acrylic as this can deteriorate the acrylic and void your warranty.

Exterior Shell & Legs

Use a gentle all-purpose cleaner, and if desired, your standard treatment room disinfectant cleaner.

Wipe gently with a slightly damp towel, then dry. If desired, use a gentle all-purpose cleaner.