

SKIN BARRIER REPAIR PROTOCOL

Treatment Time:
30 Minutes

EONIAN

LightStim⁺

Step 1: Skin Assessment

Use your protocol for skin barrier damage assessment: visual examination, tactile assessment, and/or any specialized tools to measure hydration levels or TEWL.

Step 2: Cleanse

Eye/ Lip Makeup Removal	Use a gentle cleanser - with no actives - of your choice	Remove gently and thoroughly
Full Gentle Cleanse		

Step 3: Treat

Treatment	Use 3-4 Pumps of Supreme Recovery Cream	Gently apply evenly to face and neck
-----------	---	--------------------------------------

Step 4: Treat With LightStim

LightStim	Perform LightStim treatment according to manufacturers instructions.
-----------	--

Step 5: Hydrate / Protect

Hydrate	Apply a few drops of Ethereal Effect Serum	Gently press into face and neck
Protect	Apply a mineral sunscreen of your choice	Gently apply evenly to face and neck
At home treatment	Highly recommend patient continue Supreme Recovery Cream everyday and night for 15 -30 days for barrier repair.	

HEALTHY SKIN BARRIER PROTOCOL

At Home Protocol

EONIAN

LightStim⁺

Step 1: Cleanse

Eye/ Lip Makeup Removal	Use a gentle cleanser that contains no acids (glycolic, lactic, salicylic) of your choice	Remove gently and thoroughly
Full Gentle Cleanse		

Step 2: Treat

Treatment	Use 3-4 Pumps of Supreme Recovery Cream	Gently apply evenly to face and neck
-----------	--	--------------------------------------

Step 3: Treat With LightStim

LightStim	Perform LightStim treatment according to manufacturers instructions.
-----------	--

Step 4: Hydrate / Protect

Hydrate	Apply a few drops of Ethereal Effect Serum to fingertips	Gently press into skin of face and neck
Protect	Apply a mineral sunscreen of your choice	Gently apply evenly to face and neck
AM / PM	For skin barrier repair, continue using Supreme Recovery Cream everyday and night for 15 -30 days. Your provider can assess your barrier and advise when to resume the use of acids/actives.	