

POWER PEPTIDE FACIAL WITH LIGHTSTIM

This potent, patented vitamin c facial treatment is perfect for those looking to bring back a youthful firmness and glow. A vitamin c enzyme peel, combines with a unique, pure vitamin c delivery system encourage firming and brightening. You will leave relaxed, radiant, and on your way to firmer skin.

INTENSITY LEVEL:

MEDIUM-AGGRESSIVE

TREATMENT TIME:

45-60 minutes

PRODUCTS:

CLEANSING GEL

EXFOLIATING CLEANSER

PRE-TREATMENT TONER

VITAMIN C PEEL 1

HONEY TRIZYME PEEL

POWER SERUM

FIRMABRIGHT

POLYPEPTIDE COLLAGEL FACE MASK

FIRMING MOISTURIZER

SOOTHING BALM

POWER LIFT

CHOOSE ONE:

UPLIFT EYE

EYE AUTHORITY

CHOOSE ONE:

SOLAR DEFENSE TINTED

SOLAR DEFENSE NON-TINTED

CHOOSE ONE:

PERFECTING GLOSS

LIP SHIELD

PRE-CLEANSE: 3 minutes

½ tsp. **CLEANSING GEL**

With wet hands apply a small amount and gently massage around dry lashes and the facial area

CLEANSE: 3 minutes

½ tsp. **EXFOLIATING CLEANSER**

Massage into skin focusing on areas of congestion. Remove with sponges and water.

tone: 1 minute

¼ tsp. **PRE-TREATMENT TONER**

Saturate cotton pad and wipe face to remove excess oils and prep skin. Apply eye compress. Examine skin and perform skin consultation, note any areas needing extractions.

PEEL 1: 2 minutes

¼ tsp. **VITAMIN C PEEL 1**

Begin steam (optional). Massage Vitamin C Peel for 1-2 minutes. Do not remove.

Note: Massage Only. Do not scrub vigorously.

PEEL 2: 3 minutes

¼ tsp. **HONEY TRIZYME PEEL**

Apply over Vitamin C Peel 1 and continue gently massaging for 2-3 more minutes. To achieve more exfoliation, peel combination may be left on skin for up to 6 minutes. Turn off steam. Remove with a steamed towel.

EXTRACT: 5 minutes (OPTIONAL)

¼ tsp. **PRE-TREATMENT TONER**

Saturate cotton pad and apply over areas needing extractions.

TREAT: 1 minute

⅛ tsp. **POWER SERUM +**
3-5 drops **FIRMABRIGHT**

Mix products together in hands and apply to entire face.

LIGHTSTIM: 20 minutes

USE ANTI-AGING LIGHTSTIM

Perform LightStim treatment as directed by manufacturer to enhance the benefits of the treatment.

HAND MASSAGE

½ tsp. **FIRMING MOISTURIZER +**
½ tsp. **SOOTHING BALM**

Perform during mask treatment. Warm Soothing Balm between hands. Mix with Firming Moisturizer and perform hand and arm massage.

Option: You may also incorporate an add-on service of your choice at this time.

MASK: 10 minutes

1 POLYPEPTIDE COLLAGEL FACE MASK

Unwrap and apply both sections to face beginning with lower portion. Gentle massage or pressure points may be performed over mask. Perform hand and arm massage while mask is on.

Remove by peeling mask off of skin and discarding.

EYE REPAIR: 1 minute

⅛ tsp. **UPLIFT EYE**
OR
⅛ tsp. **EYE AUTHORITY**

Uplift Eye: Lifting, Firming, Hydrating (safe for eyelid)

Eye Authority: Dark Circles, Puffiness, Crow's Feet

Apply to entire orbital area.

MOISTURIZE: 1 minute

¼ tsp. **POWER LIFT +**
3-5 drops **FIRMABRIGHT**

Mix products together in hands and apply to entire facial area.

PROTECT: 1 minute

¼ tsp. **SOLAR DEFENSE TINTED or**
SOLAR DEFENSE NON-TINTED

Apply to entire face and massage until blended.

LIP TREATMENT: 1 minute

⅛ tsp. **PERFECTING GLOSS OR**
LIP SHIELD

As an option, add a small portion of Perfecting Gloss or Lip Shield within lip line.

