

SELF GUIDED: KNESKO GEMCLINICAL® x LIGHTSTIM EXPERIENCE

ELEVATE YOUR SKIN AT THE CELLULAR LEVEL WITH A SELF-GUIDED GEMSTONE-INFUSED LIGHT THERAPY EXPERIENCE FEATURING KNESKO'S GEMCLINICAL AND LIGHTSTIM'S LED TECHNOLOGY. SUPERCHARGE YOUR SKIN AND SPIRIT FOR A GLOW THAT RADIATES FROM WITHIN....

EXPERIENCE TIME: 60 MINUTES

- 1. REMOVE MAKEUP**
- 2. HELP THE GUEST SELECT THE KNESKO COLLAGEN FACE, EYE, LIP, NECK OR DÉCOLLETÉ MASKS, AND GEMSTONE BOOSTER SERUM THAT IS BEST FOR THEIR SKIN TYPE AND CONCERNS.**
- 3. PROVIDE GUEST WITH INSTRUCTIONS TO APPLY THE MASKS IN THE RELAXATION SPACE**
- 4. APPLY KNESKO COLLAGEN MASK: WHEN APPLYING ANY MASK, MAKE SURE TO MÂSKAGE BY MASSAGING LIGHTLY OVER THE MASK. THIS HELPS TO INCREASE ABSORPTION AND RELEASE ANY AIR BUBBLES IN THE MASK. LEAVE THE MASKS ON THE SKIN FOR AT LEAST 15-20 MINUTES, DEPENDING ON THE LENGTH OF THE SERVICE.**
- 5. REMOVE THE MASKS**
- 6. TRANSITION GUEST TO THE LIGHTSTIM BED FOR 20 MINUTES LED LIGHT THERAPY TREATMENT UNDER THE PRO PANEL FOLLOWED BY 20 MINUTES FACE DOWN ON THE LED BED**
- 7. HAVE A LOCKER OR FRONT DESK ATTENDANT DISPLAY GEMSTONE BOOSTER BAR TO DISPENSE INTO GUEST'S HANDS FOR SELF-APPLICATION POST LIGHTSTIM TREATMENT.**
- 8. PULL THE GUEST'S SELECTED GEMSTONE BOOSTER(S) FROM THE SPA BOUTIQUE FOR THEM TO PURCHASE AT CHECK OUT.**