

# Le Mieux Regenerate & Renew with LightStim Anti-Aging ProPanel



Combine the power of LightStim Anti-Aging with Le Mieux growth factors to remodel aging skin and accelerate recovery post procedure. Biomimetic growth factors combined with red light boost cell renewal and collagen production to revitalize, and firm mature or environmentally damaged skin.

To learn more about Le Mieux products and protocols, visit [www.lemieuxpro.com](http://www.lemieuxpro.com).

LightStim<sup>+</sup>

## Protocol supplies

- [Bare Beauty Cleansing Oil](#)
- [Peptide Foam Cleanser](#)
- [Skin Perfecter](#)
- [AML20 Peel Solution](#)
- [Ionized Oxygen Infuser](#)
- [EGF-DNA](#)
- [Iso-Cell Recovery Solution](#)
- [O2 Calming Gel](#)
- [Hyaluronic Shea Mask](#)
- [Bio Cell Rejuvenating Cream](#)
- [Eye Wrinkle Corrector](#)
- [Just Glow BB Cream SPF 50](#)
- Light Stim Anti-Aging ProPanel

|                                      |                                                                                                                                                                                                                                                                                                                                                                                                   |
|--------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Cleanse I</b>                     | Perform first cleanse with <a href="#">Bare Beauty Cleansing Oil</a> . Apply to dry skin with dry hands and complete cleanse routine. Add water to emulsify and remove.                                                                                                                                                                                                                           |
| <b>Analyze</b>                       | Identify client's skin concerns and assess skin barrier integrity.<br><b>Complete <a href="#">Client Intake Form</a> and verify contraindications prior to treatment.</b>                                                                                                                                                                                                                         |
| <b>Cleanse II</b>                    | Lather a dime-size amount of <a href="#">Peptide Foam Cleanser</a> with water.<br>Perform cleanse, avoiding lash line. Remove thoroughly.<br>Optional: Use ultrasonic beauty tool, <a href="#">Skin Perfecter</a> , to remove cleanser.                                                                                                                                                           |
| <b>Exfoliate</b>                     | Dispense 2 full droppers of <a href="#">AML20 Peel Solution</a> into dish.<br>Apply using Peel Solutions Application Technique.<br>Avoid vermilion line of lips and orbital area. Gauge sensations.<br>If sensations spike rapidly or become uncomfortable, remove immediately.<br>Process for no more than 10 minutes. Do not use steam.<br>Remove with wet cotton until no sensations are felt. |
| <b>Serum Infusion</b>                | Apply Le Mieux Repair & Regenerate blend in <a href="#">Ionized Oxygen Infuser</a> : <ul style="list-style-type: none"><li>• Load mini-flask with <a href="#">Iso-Cell Recovery Solution</a> up to first line of flask.</li><li>• Add 3-4 drops of <a href="#">EGF-DNA</a>.</li><li>• Swirl to mix, and mist close to skin until skin is fully saturated.</li></ul>                               |
| <b>Mask &amp; Gua Sha</b>            | Mix 1 pump of <a href="#">O<sup>2</sup> Calming Gel</a> with equal parts <a href="#">Hyaluronic Shea Mask</a> .<br>Apply generously with fan brush.<br>Perform <a href="#">Meridian Gua Sha</a> technique.<br>Leave on skin for increased benefit.                                                                                                                                                |
| <b>LightStim Anti-Aging ProPanel</b> | Perform LightStim <b>Anti-Aging ProPanel</b> treatment according to manufacturer instructions.                                                                                                                                                                                                                                                                                                    |
| <b>Finish</b>                        | Moisturize and protect:<br>Apply <a href="#">Bio Cell Rejuvenating Cream</a> .<br>Apply a rice grain size of <a href="#">Eye Wrinkle Corrector</a> to orbital area.<br>Finish with <a href="#">Just Glow BB Cream SPF50</a> .                                                                                                                                                                     |