

Le Mieux Sensitivity & Post-Procedure with LightStim Anti-Aging ProPanel

Combine the power of LightStim Anti-Aging with Le Mieux growth factors to strengthen delicate skin and accelerate recovery post procedure. Biomimetic growth factors combined with red light boost cell renewal and enhance skin's natural repair cycle.



LightStim⁺

To learn more about Le Mieux products and protocols, visit www.lemieuxpro.com.

Protocol supplies

- [Bare Beauty Cleansing Oil](#)
- [Peptide Foam Cleanser](#)
- [Ionized Oxygen Infuser](#)
- [EGF-DNA](#)
- [O2 Calming Gel](#)
- [Derma Relief Serum](#)
- [Bio Cell Rejuvenating Cream](#)
- [Eye Wrinkle Corrector](#)
- [Just Glow BB Cream SPF 50](#)
- LightStim Anti-Aging ProPanel

Cleanse I	Perform first cleanse with Bare Beauty Cleansing Oil . Apply to dry skin with dry hands and complete cleanse routine. Add water to emulsify and remove.
Analyze	Identify client's skin concerns and assess skin barrier integrity. Complete Client Intake Form and verify contraindications prior to treatment.
Cleanse II	Lather a dime-size amount of Peptide Foam Cleanser with water. Perform cleanse, avoiding lash line. Remove thoroughly.
Serum Infusion	Apply Le Mieux Repair & Regenerate blend in Ionized Oxygen Infuser : • Load mini-flask with Iso-Cell Recovery Solution up to first line of flask. • Add 3-4 drops of EGF-DNA. • Swirl to mix, and mist close to skin until skin is fully saturated.
Mask	Apply 1 pump of O² Calming Gel generously with fan brush. Leave on skin for increased benefit or remove excess.
LightStim Anti-Aging Pro Panel	Perform LightStim Anti-Aging ProPanel treatment according to manufacturer instructions.
Finish	Moisturize and protect: • Apply 1-2 drops of Derma Relief Serum. Apply Bio Cell Rejuvenating Cream . Apply a rice grain size of Eye Wrinkle Corrector to orbital area. • Finish with Just Glow BB Cream SPF50