
NECESSARY PRODUCTS:

DIAMOND EXPERIENCE LAVENDER NEURO-AROMA
DIAMOND LUMINOUS EXPERTISE RICH LUXURY CLEANSE
DIAMOND LUMINOUS EXPERTISE CLARITY TONING LOTION
GLYCO 3 PEEL/RADIANCE LIFTING PEEL
DIAMOND LUMINOUS BRIGHTENING CONCENTRATE
DIAMOND EXPERIENCE ESSENCE
DIAMOND LUMINOUS PERFECTING GLOWING MASK
DIAMOND EXTREME EYE
DIAMOND COCOON SHEER CREAM/DIAMOND LUMINOUS UV DEFENSE

CONCENTRATE PREPARATION:

For one dose: Mix two measuring spoons of DIAMOND LUMINOUS EXPERTISE BRIGHTENING CONCENTRATE in a bowl with two pumps of DIAMOND LUMINOUS EXPERTISE CLARITY TONING LOTION. Blend with a brush until an even texture and light foam is achieved.

PERFORM INHALATIONS AND BODY ALIGNMENT:

Assist the guest with deep breathing through Inhalations with DIAMOND EXPERIENCE LAVENDER NEURO-AROMA.

NB CLEANSING:

CLEANSING

Using a cotton pad, clean the eye and lip contour area with DIAMOND LUMINOUS EXPERTISE RICH LUXURY CLEANSE or NB CEUTICAL EYE & LIP MAKEUP REMOVER.

Apply DIAMOND LUMINOUS EXPERTISE RICH LUXURY CLEANSE on face, neck and décolleté using NATURA BISSÉ CLEANSING TECHNIQUE.

(see illustration)

Remove the product with the NATURA BISSÉ cleansing sponges moistened in warm water.

Apply DIAMOND LUMINOUS EXPERTISE CLARITY TONING LOTION. Dry Skin

EXFOLIATION:

GLYCO 3 PEEL

Apply G3P phase 1. Leave on for 10 minutes. Do not remove

Apply G3P phase 2. Create a uniform white layer with G3P phase 2 (powder) by pressing the product lightly onto the skin with a non-latex

makeup sponge. (Remember to cover the eyes with a dampened cotton pad and to inform the client before pressing gently on the nostrils). Do not remove powder.

Apply G3P phase 3 with the included brush to create an effervescent effect on the skin. Follow with a soft, digital massage.

Remove with warm water and NATURA BISSÉ facial cleansing sponges.

Or

RADIANCE LIFTING PEEL

MULTIACTIVE PEEL

Arrange the treatment table so the guest's head is higher than the rest of the body.

Starting from the forehead, apply MULTIACTIVE PEEL with a brush on face, neck, and décolleté.

Make sure to remove any excess from the brush to avoid dripping.

Leave on exposition during 5 minutes (exposition time starts when application starts).

During the exposition time, perform pressure points on arms, back and cervical area (BALANCING CRANIAL MASSAGE - 1st step).

BRIGHTENING BOOSTER

Without removing the MULTIACTIVE PEEL, apply the COMPRESSED FACE MASK soaked in BRIGHTENING BOOSTER.

Apply the rest of the BRIGHTENING BOOSTER with a brush on the area without mask and leave it on for 5 minutes.

During the exposition time, perform pressure points on head (BALANCING CRANIAL MASSAGE - 2nd step).

Remove with warm water and NATURA BISSÉ facial cleansing sponges. Dry the skin.

CONCENTRATE

Step 1 Apply the prepared concentrate with a brush with "eraser" technique, only on hyperpigmented areas, until completely absorbed.

Step 2 Extend the product using digital maneuvers on face, neck and décolleté. If needed, remove any excess powder residue.

ELIPSA INSTRUCTIONS FOR USE

1. Plug into a wall outlet
2. Press any of the buttons to wake up the device if the time display is off
3. Select the treatment mode indicated by the color on the time display. Select Red – Wrinkles
4. Select treatment time by using the 5min button
5. Select the power button to start treatment
6. Light will turn off once treatment is complete

ELIPSA TREATMENT ROOM PROTOCOL

1. Position LightStim provided goggles over the client's eyes and ensure the elastic band is in place, snugly but comfortably around the head – if necessary moistened cotton rounds may be used under the goggles
2. Before moving Elipsa into position over the face, turn the device on – this will provide an opportunity for the eyes to adjust to the brightness (see instructions for use above)
3. Position Elipsa approximately 1" from the treatment area by using the adjustable legs to increase or decrease the height
4. When the treatment is complete, remove the device
5. Continue with the following steps of the protocol

NB MASSAGE:

Apply DIAMOND EXPERIENCE ESSENCE to the face, neck, and décolleté, working the product using NATURA BISSÉ LIFTING MANUEVER MASSAGE (see illustration).

Continue with SHIATSU MASSAGE

Perform repeated pressure movements perpendicular to the skin for 3-5 minutes, gradually increasing the pressure exerted, holding it for a second longer and then slowly reducing the pressure.

After completely releasing the pressure and without lifting your finger from that point, apply pressure to the same place.

Repeat this maneuver 3 times in each point.

NB MASK:

Apply a layer of the DIAMOND LUMINOUS EXPERIENCE PERFECTING GLOWING MASK to the face, neck, and décolleté. leave on for 15-20 mins. During exposition time of the mask, perform a hand and arm massage.

PROTECTION AND PREVENTION:

Apply DIAMOND LUMINOUS CLARITY TONING LOTION onto the face, neck and décolleté. DIAMOND EXTREME EYE to the eye contour area.

Finish by applying DIAMOND COCOON SHEER CREAM/DIAMOND LUMINOUS UV DEFENSE

FINISH:

Perform final BODY ALIGNMENT