



Advanced LED and PCA SKIN® Professional Treatment for Aging Skin: This advanced LED and professional peel treatment will provide immediate smoothing, brightening and hydrating benefits to your patients' skin. This protocol is most appropriate for Fitzpatrick skin types I – V, concerned with fine lines and wrinkles, laxity and surface texture.

Cleanse

1. **Facial Wash Oily/Problem** – Cleanse the area of treatment thoroughly, remove with warm water and pat dry.

Prep

2. **Nutrient Toner** – Apply with a cotton pad to prep the skin. Allow the skin to dry. This product may sting or tingle on some individuals. Assess sensitivity by asking the patient, "On a scale of one to ten, ten being extremely active, how do you rate this sensation?"

Pre-Treat

3. **Hydrate: Therapeutic Oat Milk Mask** – For additional hydration and product penetration, apply a thin, even layer to the area of treatment. Allow the mask to remain on the skin for up to ten minutes. Massage (if desired) using gentle, but firm, pressure. Remove thoroughly with warm water on cotton pads or warm towels. If needed **Nutrient Toner** may be applied.

Treat: LED

4. **LightStim** – Administer the treatment using LightStim as indicated for rejuvenation and age control.

Treat: Professional Peel

5. **Ultra Peel®** – Before using, gently agitate the product. Apply the solution to a cotton pad, placing the neck of the bottle directly onto the pad, spacing it to give a sufficient working area to complete a full-face treatment (about four to five spots on the pad). Use the scale of 1 to 10 to gauge sensitivity. Maximum recommended layers is 2 with a cotton pad when incorporated with LED.

Advanced LightStim and PCA SKIN® Professional Treatment for Aging Skin (Continued)

Correct

6. **ExLinea® Pro Peptide Serum** – Spot treat areas of laxity and wrinkling.
7. **Pro-Max Age Renewal** – Apply one to two pumps to the treatment area to strengthen the ECM and increase hydration.
8. **C&E Advanced** and **Hyaluronic Acid Boosting Serum** – Blend one to two pumps of the **Hyaluronic Acid Boosting Serum** and a pea-sized amount of the **C&E Advanced** to the treatment area to brighten, lighten and tighten the skin as well as increase hydration.
9. **Dual Action Redness Relief** – Apply one to two pumps to the area of treatment to reduce inflammation.

Retinoid Booster (Optional)

10. **Advanced Treatment Booster** – Dispense two pumps into gloved hands. Gently spread the product over the area of treatment, keeping application even and thin.

Hydrate and Protect

11. **ReBalance** and **Hydrator Plus Broad Spectrum SPF 30** – Complete the treatment with this combination to calm, hydrate and protect the skin from UVA and UVB exposure.