



LightStim⁺

Advanced Dermaplaning with LED, Enzyme and Antioxidant Infusion – This summer friendly treatment safely incorporates enzymes and antioxidants with dermaplaning to optimize smoothing of the surface and protection from environmental stressors. It is appropriate for all Fitzpatrick skin types and conditions.

Cleanse

1. **Facial Wash Oily/Problem** – Cleanse the area of treatment thoroughly, remove with warm water and pat dry.

Prep

2. **Nutrient Toner** – Apply with a cotton pad to prep the skin. Allow the skin to dry. This product may sting or tingle on some individuals. Assess sensitivity by asking the patient, “On a scale of one to ten, ten being extremely active, how do you rate this sensation?”

Enzymatic Pre-treatment

3. **Revitalize: Therapeutic Papaya Mask** – Apply a generous amount to the area of treatment using gloved hands, enough to cover the face. Use the scale of 1 to 10 to gauge sensitivity. Allow the mask to remain on the skin for up to 5 minutes when using Revitalize. Remove completely with cotton pads moistened with warm water.

Treat

4. **Dermaplaning** – Perform dermaplaning. Before moving on to the next step, be sure the skin is clean and dry.

Professional Tip: Detoxifier (Step 2 of the Oxygenating Trio®) may be applied if preferred to do a wet treatment on someone who may have severely dry/compromised skin. It is a professional’s preference when used.

Note: If wet dermaplaning was performed apply Nutrient Toner to a cotton pad and apply to the skin to remove any excess product remaining on the surface of the skin. Before moving on to the next step ensure the skin is completely dry.

Treat: LED

5. **LightStim** – Administer the treatment using **LightStim** or your preferred LED device as indicated for rejuvenation and age control.

Treat

6. **Enzymatic Treatment** – Dispense a generous amount into a small peel dish.
 - Apply with a mask brush over the entire area of treatment. Massage into the skin for approximately three to five minutes or until fully absorbed.
 - If there is excess product on the skin, apply Nutrient Toner to a cotton pad and lightly remove.
 - The client may feel some stimulation during the treatment. Fan the client to keep them cool.

Correct

7. **Pro-Max Age Renewal** – Apply one to two pumps to fight and prevent signs of aging by utilizing Micro Growth Factor Technology that has been proven to increase skin fullness by 60%.
8. **C&E Advanced** and **Vitamin b₃ Brightening Serum** – Apply a pea-sized amount of C&E Advanced with one two pumps of Vitamin b₃ Brightening Serum to protect from free radical damage and plump and firm the skin through increased hydration.

Hydrate and Protect

9. **ReBalance** and **Daily Defense Broad Spectrum SPF 50+** – Complete the treatment with this combination to calm, hydrate and protect the skin from UVA and UVB exposure.

Added Protection

10. **Daily Defense Mist** – For added protection against environmental stressors hold the spray 8 – 10 away from face and mist directly onto skin.

Post-Procedure Care

It is recommended to use the **Post-Procedure Solution** for 3 – 5 days post-treatment or until peeling has resolved. The products included in this kit are designed to protect and heal the barrier after any superficial professional peel treatment.

Pro Tips

- Incorporate the **Overnight Lip Mask** into any regimen for softer, smoother lips in the morning.
- Once skin has returned to baseline to prolong and enhance treatment results, incorporate our **Hyaluronic Acid Overnight Mask**. This mask has been proven to increase skin hydration by 75% when applied once and left on the skin for eight hours.