

# Saian Ultimate Signature Facial Protocol

## Treatment

This unique treatment infuses collagen and elastin nano-molecules into the skin, tightens the muscles with microcurrent, and revitalizes the skin with peptides and LED. The result is younger and suppler skin just after one treatment.

## Step-By-Step Product Application

1. Cleanse the neck and face with Saian Replenishing Cleanser or Strawberry Cleanser, depending on skin type.
2. Blend a teaspoon of Saian Enzyme Mask powder with equal part warm water to make a smooth paste, and apply to the face, neck and décolleté for 7-10 minutes with steam.
3. Remove the Saian Enzyme Mask thoroughly.
4. Spray the face with Saian Herbal Toner to help bring the pH back to normal.
5. Use a stiff fan brush to saturate the face with water, then cleanse the face with Saian Professional Strength Ultrasonic Spatula on the high setting (H) for 3-5 minutes. Make sure the skin is always wet while working with the device, and point the "blade" downwards at a 45 degree angle when in contact with skin.
6. Perform extractions and follow with Saian High Frequency, if needed.
7. Perform facial massage with Essence by Saian Organic Beauty Oil. (optional)
8. Remove the massage oil thoroughly.
9. Apply Saian Botanical Stem Cell Mask to the skin in a thin layer avoiding the hairline and eyebrows/lashes. You will need approximately 2 pumps.
10. After applying the mask, cover the client's eyes with protective goggles (optional for comfort) and position your LightStim anti-aging panel about an inch away from the client's face. Leave the mask under the light for 15-20 minutes for best results. The panel has a timer, and will shut off automatically.
11. Remove the Saian Botanical Stem Cell Mask thoroughly.
12. After removing the mask, open the Saian Hyaluronic Acid Ampoule according to the directions on the box, making sure to wrap the top in a tissue before snapping it off. Pour the desired amount of serum into the palm of your hand, and apply to the face, neck and décolleté. Gently pat with open palms, especially around the wrinkle-prone areas.
13. Spray the face, neck and décolleté with Saian Pure Collagen Elastin Spray 4-6 times.
14. Apply a dime-sized amount of Saian Neck and Chest Lift Crème to the neck, jowls and décolleté.
15. Fold the Saian Pure Collagen Sheet Mask in half over the client's face. Measure the distance from hairline to nostrils. Mark and cut the mask to expose the nostrils.
16. Apply the collagen mask immediately over the face.
17. Spray with Saian Pure Collagen Elastin Spray again over the mask 4-6 times just until it is moist.
18. Dip sponges into warm water, squeeze out excess water and press with moist sponges over the mask to mold it to the face, making sure there are no air bubbles.
19. Use the Saian Skin Pro 2 portable galvanic on the "nutrition" mode rolling over the mask to increase absorption for 4 minutes. The machine has an automatic timer and will shut off after each 4 minute step.
20. Cover the mask with a warm towel, or turn on the steam over it for an additional 15 minutes.
21. Remove the collagen sheet by lifting it off in one piece.
22. Apply 2 drops of Saian Tri-Active Collagen Serum to the whole face focusing on wrinkle-prone areas.
23. Apply 2 drops of Saian Pure Hyaluronic Acid Serum to the whole face focusing on wrinkle-prone areas.
24. Apply Saian Active Renewal Crème (weightless, fragrance free) or Saian Amino-Lift Leave-On Mask (richer, antioxidant w/neroli oil) to maintain moisture.
25. Roll over the face with previously frozen Saian Cool Duo Massagers for 2-3 minutes to close the pores.
26. Spray with Saian Sunscreen Mist SPF 30.

## Frequency

For best results, treatments should be performed once in 3 weeks, a series of 6 is encouraged.

## Suggested Retail For Home Care

- >> Saian Tri-Active Serum
- >> Saian Active Renewal Serum (best seller)
- >> Saian Active Renewal Crème (best seller)
- >> SNC Labs by Saian Hyaluronic Acid Supplements

