

LightStim⁺

Clarifying Protocol

A gentle and effective treatment that utilizes bioactive peptides and LED light therapy, focusing on soothing and clarifying the complexion. This protocol is designed to offer care for skin experiencing irritation or blemishes. Perfect for soothing, addressing redness, and fortifying the skin's natural defense barrier.

Step 1: Pre-Cleanse

- **Purifying Cleanser:** Thoroughly cleanse for 1-2 minutes to remove existing eye and face makeup if applicable. Remove with wet sponges or a steamed towel. Pre-cleansing is not required for all clients.

Step 2: Cleanse

- **Balancing Cleanser:** Thoroughly cleanse for 1-2 minutes. Remove with wet sponges or a steamed towel.

Step 3: Exfoliate

- **Lactic 5% + Collagen:** Apply 4-6 drops directly to cleansed, dry skin. Let self neutralize for 5-10 minutes. Do not remove.

Step 4: Extractions (optional)

- Perform extractions if necessary

Step 5: Peptide Treatment*

5a: PEPTI-clear

- **PEPTI-clear:** Apply 4-6 drops to the treatment area. Allow 1-2 minutes for complete product absorption.

5b: PEPTI-repair

- **PEPTI-repair:** Apply 4-6 drops to the treatment area. Allow 1-2 minutes for complete product absorption.

5c: Medium MW Hyaluronic Peptide Serum

- **Medium MW Hyaluronic Peptide Serum:** Dispense 1 packet to the treatment area. Massage into skin for 1-2 minutes for complete product absorption.

***Additional restore products may be utilized during the peptide treatment phase to address other specific skin concerns**

Step 6: Mask

- **PEPTI-Nano Mask:** Apply according to packaging directions and adjust to facial contour. Leave on during LED treatment. Re-apply remaining serum every 2-5 minutes with a fan brush.

Step 7: LED Treatment with LightStim

- Perform LED treatment in accordance with manufacturer's instructions.

Step 8:

- Remove mask.

Step 9: Peptide Treatment*

- **Rejuvenator Peptide:** Massage a dime-sized amount into the skin of the treatment area, utilize the appropriate rejuvenator strength for your client.

***Some users of this product may experience minor tingling, which should subside shortly after application. If the sensation persists, please rinse and cleanse the face.**

Step 10: Moisturize

- **Collagen Gel:** Apply and gently massage into skin until completely absorbed. Any moisturizer from the protect phase may be utilized in this step.

Suggested Home Care Products:

Balancing Cleanser AM and PM

PEPTI-clear AM and PM

PEPTI-repair AM and PM

PEPTI-hyaluronic AM and PM

Rejuvenator Peptide AM and PM*

Collagen Gel AM and PM

***Recommend strength based on the appropriate Rejuvenator formula for client's skin health.**